


Maintenance Tips

HIGH SPEED HANDPIECE MAINTENANCE GUIDELINES

AFTER EACH PATIENT

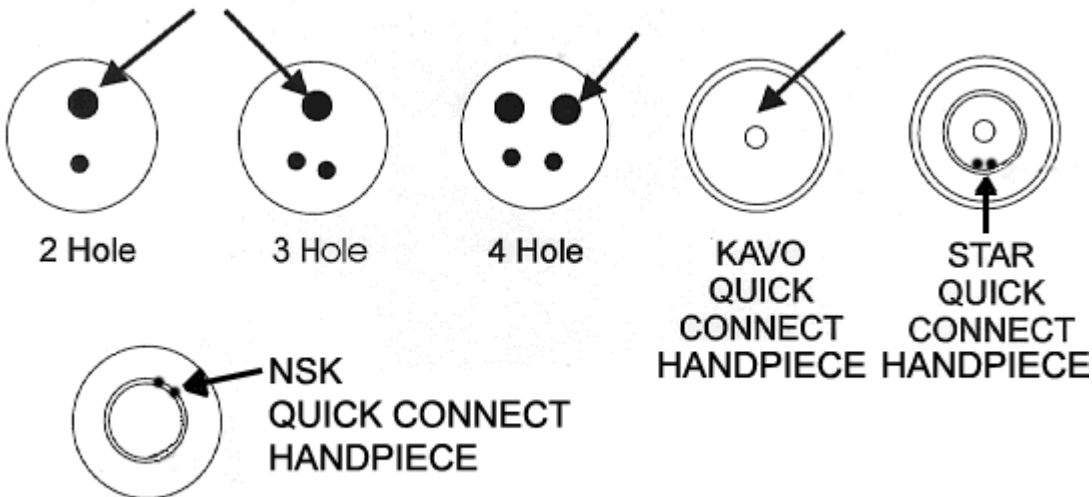

Remove handpiece from unit tubing, clean with alcohol (Do Not use harsh cleaning agents, Do Not place in ultra sonic unit).

With bur inserted, place lubricant (2 or 3 drops) into air drive port, - at the same time, rotate the bur between your fingers forward/backward 1/4 turns for a few seconds.

Re-connect handpiece to unit tubing, place an autoclave bag over the handpiece & run 30 seconds to eliminate excess oil from the turbine. (This procedure is primarily to clean the handpiece not lubricate it).

Remove from unit tubing, and autoclave.

After autoclaving, allow to cool gradually, about 30 minutes, to room temperature.

With bur in handpiece place lubricant (2 or 3 drops) into air drive port, - at the same time, rotate the bur between your fingers forward/backward 1/4 turns for a few seconds.

Connect handpiece to unit tubing & run for 30 seconds to eliminate excess oil from the turbine (this procedure is to lubricate the handpiece).


NOTE: A liquid lubricant (oil) seems to work best.

>>>>>>SET AIR PRESSURE TO MANUFACTURE'S SPECIFICATIONS<<<<<<<<

LUBRICATING CONTRA ANGLES AND HEADS

AFTER EACH PROCEDURE

1. Disassemble head from Contra Angle as shown in diagram.
2. Wipe off parts with alcohol, NEVER place any parts in an ultra sonic unit!

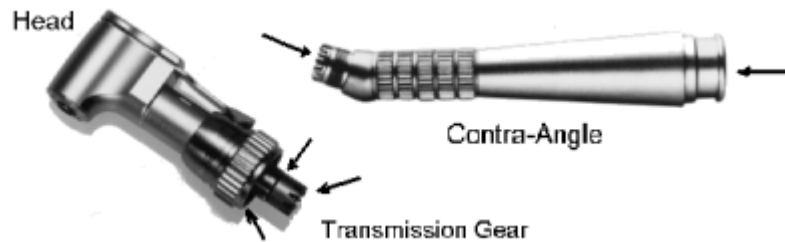

3. Place a lubricant (2 or 3 drops) into all parts noted by the arrow.
4. Reassemble, the Contra Angle and place it on a Slow speed motor and run for 30 seconds (this is for cleaning purposes).
5. Autoclave and let cool.
6. Repeat steps 1 thru 4 (this is for lubricating purposes).